## DENTAL DAM CLAMP INSTRUCTIONS FOR USE

**Warnings** – Intended for professional dental use only. Clamps should be replaced if distorted, worn or cracked. The life of the clamp is dependent upon the number of uses and care taken during application, cleaning and storage. Clamps can become damaged if pH neutral detergent or cleaning solutions are not used. Clamps are known to break or become damaged after multiple uses or as a result of over extension.

**Intended Use** – The clamp is intended to be placed over a tooth to secure the dental dam in place and to clear the tissue from the procedural area.

**Instructions for Use** – Remove clamp from packaging, then clean and sterilize clamp prior to use. Tie safety ligation (dental floss) around the clamp to prevent swallowing should the clamp become dislodged or break. (See Illustration A for recommended technique). Pre-fit the clamp to ensure that there is a four-point contact with the tooth's proximal surfaces. Using the Winged Technique, a clamp is then placed into the pre-punched hole in the dam so that its wings hold the hole open. With the aid of the clamp forceps the dental dam and clamp can be placed over the tooth. The dental dam is then slipped off of the wings of the clamp and onto the tooth.

Illustration A

**Cleaning** – Clamps should be rinsed and cleaned immediately following the procedure. Failure to clean will decrease the life of the clamp and can result in staining or corroding. Rinse and remove excess materials prior to ultrasonic cleaning. Use a pH neutral cleaning solution, such as, but not limited to BioSonic UC30 General Purpose or UC31 General Purpose with Rust Inhibitor, and follow the instructions provided by the manufacturer of the ultrasonic cleaner. Rinse cleaning agents from clamps. Inspect to confirm that all materials have been removed from clamp. Repeat if necessary. Allow clamps to dry. Refer to P/N 20004841 Dental Dam Accessories Cleaning IFU for detailed cleaning instructions.

**Sterilization** – Important, if any material is left on the clamp, it may hinder sterilization of the clamp.

Autoclave (Steam Sterilizer) for 15 minutes at 132°C/270°F in gravity autoclave.

or

Autoclave 4 minutes at 132°C/270°F in a pre-vacuum sterilizer

Follow autoclave manufacturer's instructions to ensure that maximum load is not exceeded.

**Inspection** – Always inspect the clamp for proper tension, distortion, wear or damage before use. Discard clamps if any deficiency is noted.

**Care** – To ensure maximum usage, do not bend or distort the clamp. Do not let clamps get scratched by other clamps or instruments.

When using obturation techniques involving sodium hypochlorite, ensure clamps are rinsed immediately with water after the procedure. Prolonged contact with sodium hypochlorite can cause pitting of the clamp which results in a significantly shorter clamp life.

Following these instructions will help extend the life of the clamp. It is the responsibility of the user to see that these steps and precautions are taken.

**Notice: Report any serious incident that has occurred in relation to this device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.**

Coltène/Whaledent Inc.   
235 Ascot Parkway  
Cuyahoga Falls, OH 44223 / USA  
Tel. USA & Canada 1 800 221 3046  
+1 330 916 8800  
Fax +1 330 645 8704  
info.us@coltene.com

Coltène/Whaledent GmbH + Co. KG  
Raiffeisenstrasse 30  
89129 Langenau / Germany  
Tel. +49 (0) 7345 805 0  
Fax +49 (0) 7345 805 201  
info.de@coltene.com

www.coltene.com

Coltène/Whaledent AG  
Feldwiesenstrasse 20  
9450 Altstätten / Switzerland  
Tel. +41(0)71 757 53 00  
Fax +41(0)71 757 53 01  
info.ch@coltene.com

# Recommended Cleaning, Sterilization and Maintenance of Hygienic Instruments

Dental Dam Clamps / Dental Dam Forceps / Dental Dam Punch / Dental Dam Frame (Plastic and Metal) (ISO 17664)

## CAUTION – INTENDED FOR PROFESSIONAL DENTAL USE

**ONLY.** Dental Dam Clamps, Forceps, and Frames are classified as Critical or Semi-critical instruments by the American Dental Association with reference to the Centers for Disease Control classification system due to contact with the bloodstream, normally sterile tissue, mucous membranes, or the skin. Inspect, clean and sterilize these instruments before each use, or replace them when they become distorted, worn or cracked.

## Presoak/Holding

Rinse and clean immediately following the procedure. If products cannot be cleaned immediately, soak instruments in a neutral pH cleaning solution like BioSonic UC30, UC31 or UC32.

## Cleaning

Always wear appropriate Personal Protective Equipment (PPE) while handling used dental equipment. It is important that all blood, dried body fluids and tissues be removed from the instruments prior to sterilization.

**Rinsing** – Immediately after use, rinse organic materials off instruments under warm running water. The purpose of this step is to remove most blood, fluids and tissue from the instrument.

**Ultrasonic Cleaning (recommended):** Follow ultrasonic cleaner manufacturer's instructions. Rinse and remove excess material prior to ultrasonic cleaning. Use a pH neutral, general purpose cleaning solution such as, but not limited to BioSonic UC30, UC31 or UC32. All instruments should be completely submerged in the cleaner bath. Hinged instruments should be cleaned in the open position. For cleaning times, follow the instructions listed on the cleaning solutions label. Ultrasonic cleaning solutions should be changed at least daily and more often if the solution in the tank becomes visibly dirty. Rinse cleaning agents from instruments. Inspect to see that all debris has been removed from instrument. Repeat if necessary. Allow instrument to dry.

**Manual Cleaning:** Use a stiff nylon cleaning brush to carefully scrub instrument. Use of a stainless steel wire brush or steel wool will damage the instruments. Use a pH neutral cleaning solution such as, but not limited to BioSonic UC30, UC31 or UC32. Rinse cleaning agents from instruments. Inspect to see that all debris has been removed from instrument. Repeat if necessary. Allow instrument to dry.

**Notice: Report any serious incident that has occurred in relation to this device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.**

Coltene/Whaledent Inc.   
235 Ascot Parkway  
Cuyahoga Falls, OH 44223 / USA  
Tel: USA & Canada 1 800 221 3046  
+1 330 916 8800  
Fax: +1 330 645 8704  
info.us@coltene.com

Coltene/Whaledent AG  
Feldwiesenstrasse 20  
9450 Altstätten / Switzerland  
Tel. +41(0)71 757 53 00  
Fax +41(0)71 757 53 01  
info.ch@coltene.com

Coltene/Whaledent GmbH + Co. KG  
Raiffeisenstrasse 30  
89129 Langenau / Germany  
Tel. +49 (0) 7345 805 0  
Fax +49 (0) 7345 805 201  
info.de@coltene.com

 **COLTENE**

## Sterilization

Follow autoclave manufacturer's instructions and ensure that maximum load is not exceeded. Lubricate hinged instruments with a non-silicone water soluble lubricant. Disposable sterilization pouches are recommended. For Dental Dam Clamps, Forceps and Punches, steam sterilize for 15 minutes at 132°C / 270°F in a gravity autoclave or for 4 minutes at 132°C / 270°F in a pre-vacuum sterilizer. Do not heat metal based instruments above 177°C/350°F. For Dam Frames, sterilize for 4 minutes at 132°C /270°F in a pre-vacuum sterilizer. Do not heat above 140°C/ 284°F as warping may occur. Allow instrument to dry. Make sure instruments are dry before storing.

## Trouble shooting

**Stains on Instruments** - From time to time, instruments discolor during the cleaning process. Often times this is misinterpreted as rust. Black stains are indicative that the pH of the cleaning solution was too low (acidic). Brown stains are indicative that the pH of the cleaning solution was too high (basic). Proper rinsing and drying of the instruments prior to autoclaving should eliminate any chance of staining. Dissimilar metals should be cleaned and autoclaved separately as cleaning or autoclaving together can cause galvanic staining where the two instruments were touching.

For most tabletop autoclaves, the door must be cracked about ¼ inch at the end of the cycle to allow the instruments to efficiently dry. If the door is opened too much, cold air will rush in and cause condensation on the instruments. This can lead to water stains or wet packs.

**Chemical / Cold Sterilization is not recommended for Hygienic Dental Dam instruments like clamps, forceps and punches as the 6 to 10 hour immersion times can cause corrosion and dulling of the punch and premature failure of the clamp and forceps. Additionally, soaking dam clamps in bleach can cause premature failure of the clamps.**

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