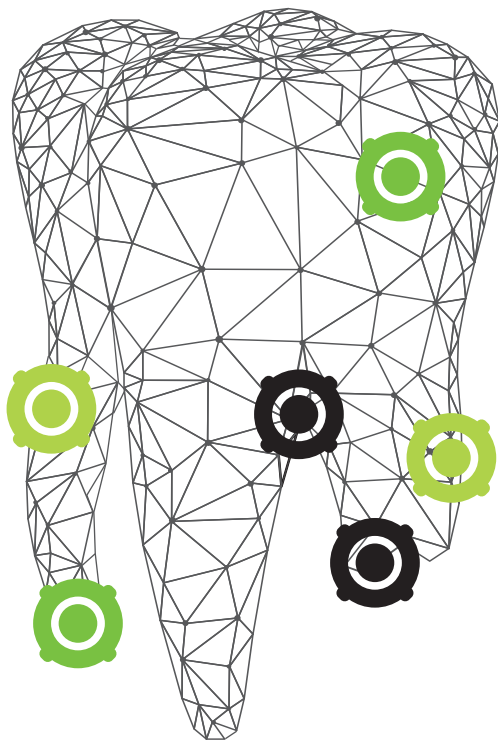
MTA SMART^{SYSTEM}

Training set

Endodontic training set for application MTA

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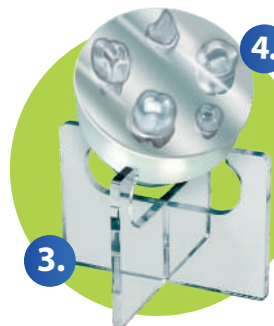
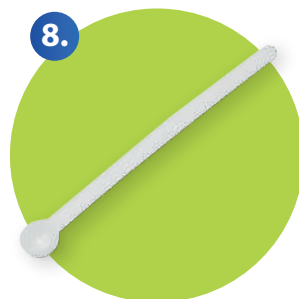
6. Mixing papers
/ Papierki do zarabiania

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8.

8. Measuring cup for MTA powder
/ Miarka do proszku

ATTENTION! Product is for training use only. It is not suitable for use on patients.

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Training teeth:

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2633-101-PCL

Maxillary first molar, 3 canals, 3 roots, perforation of the pulp chamber floor, lateral wall perforation of the palatal root

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2111-1A2-PCL

Maxillary central incisor, 1 canal, 1 root, incomplete root development

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2633-102-PCL

Maxillary first molar, 3 canals, 3 roots, perforation of the pulp chamber floor, external apex resorption

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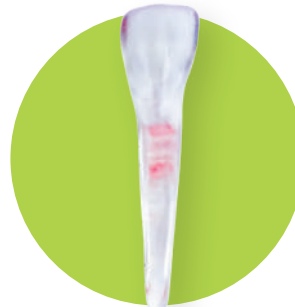
3623-101-PCL

Mandibular central incisor, 1 canal, 1 root, perforation of the pulp chamber floor

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Use of the set:

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1

Prepare the tool of your choice for handling and applying the training MTA – applicator, plugger, flat plastic.

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2

Switch on the backlight in the silicone training mass (3).

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**3.1**

Place a case-specific amount of the training MTA powder (1) on the glass slide (5).

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Use the measuring cup (8) provided to the set to measure out a portion of the powder (1), and to apply the liquid (2), use the pipette (7).

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3.3

Knead the training MTA material by mixing it for about 30 seconds until it has the consistency of soft plasticine. If the material is too thick or crumbly, add another drop of the liquid (2) and re-knead the material.

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4

Using the tool of your choice, collect the prepared training material from the glass slide (5).

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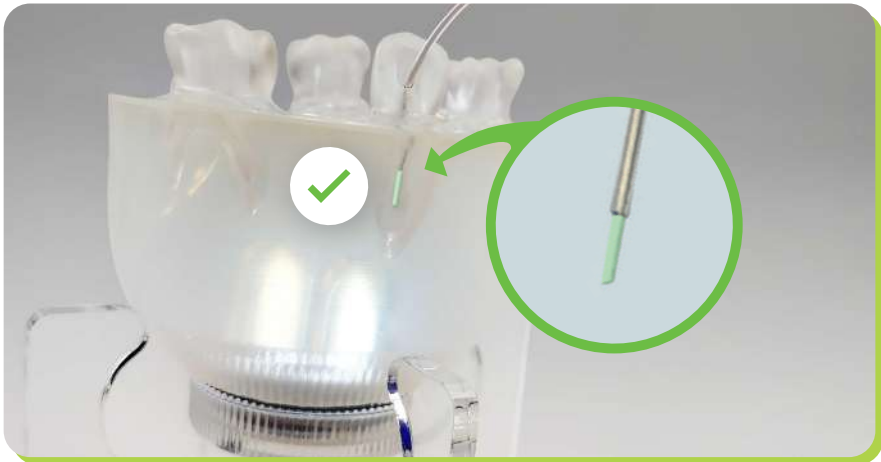
5

Place and apply the collected training MTA material to the desired area of the training tooth (4).

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6

Immediately after finishing the application, clean the glass slide (5) of the residual material and wash the glass slide (5). Clean the used tool for handling and application of the training MTA material as recommended by the manufacturer of the tools.

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Precautions:

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Contains calcium oxide. Causes serious eye damage. Wear protective gloves/protective clothing/eye protection/face protection. Skin contact: take off contaminated clothes. Wash the exposed skin areas thoroughly with soap and water for at least 10 minutes. In case of worrying symptoms consult a doctor.

If ingested: do not induce vomiting. Rinse mouth with water. Never give anything by mouth to an unconscious person, contact a doctor. Inhalation: take the injured person outside, provide warmth and peace. In case of worrying symptoms consult a doctor.

Dispose of the contents/container into properly labelled waste bins in compliance with national regulations.

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Storage conditions:

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Store only in original, sealed containers in a dry, cool and well-ventilated place. Avoid high temperatures, protect from moisture.

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MTA SMART SYSTEM
Training set

The logo features the letters 'MTA' in a bold, green, sans-serif font. To the right of 'MTA', the word 'SMART' is written in a smaller, black, sans-serif font, followed by 'SYSTEM' in an even smaller font. A stylized black graphic of a dental handpiece is positioned between 'SMART' and 'SYSTEM'. Below this, the words 'Training set' are written in a green, sans-serif font.

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