

Perioflush

for the irrigation of periodontal pockets

Liquid

D L S DENTAL LIFE
SCIENCES

Perioflush is a liquid formulated for the irrigation, rinsing and cleaning of inflamed periodontal pockets and the oral mucosa.

- dissolves and dilutes the contents of inflamed periodontal pockets
- removes 99% of microorganisms
- does not stain teeth or irritate the soft tissues of the mouth
- eliminates the inflammation of periodontal pockets and other places which are difficult or impossible for the patient to clean at home, e.g. around bridges or teeth with operculae caused by incomplete tooth eruption

INDICATIONS

- periodontal pockets caused by periodontitis or periimplantitis
- areas around overhanging fillings
- interdental spaces with inadequate contact points
- after scaling and curettage procedures to promote healing
- under gaps in permanent restorations and in areas around implants

PRESENTATION

Perioflush is available as a single pack, containing one 3 ml syringe of liquid solution together with a syringe applicator (pre-bent needle tip), or as a kit of 10 syringes (3 ml each) with 10 applicators. One syringe is to be used for one patient.

METHOD OF USE

Perioflush is a safe preparation and it may be used freely according to individual needs.

Most people (85%) achieve a considerable improvement after a single application to the infected pocket. The syringe and the applicator have to be inserted with care so as not to further damage the gingiva. Slowly press the plunger of the syringe to inject the preparation into the pocket, moving the tip along the surface of the tooth. After irrigating the pocket, rinse the mouth with water. In acute conditions or in the case of sensitive teeth, slightly heat the preparation prior to use, by placing it in, for e.g. warm water. Repeat rinsing procedure once in 3-4 months to avoid reoccurrence of the acute inflammation.

CONTRAINDICATIONS

Hypersensitivity to any of the ingredients.

INTERACTIONS

None known.

SIDE EFFECTS

None if the preparation is used in accordance with the instructions.
Perioflush can be used in patients with ulcers, diabetes and in pregnant women.

ADDITIONAL INFORMATION

For use by dental surgeons.
Store at room temperature.
Keep out of reach of children.

FORMULATION

Aqua purificata, silver nanocolloid solution, sodium nitrate, aromas, phosphoric acid, lactic acid.

Perioflush is used for irrigation of periodontal pockets:


Periodontal pockets




Around teeth with operculae, especially wisdom teeth



Under restorations such as bridges and crowns

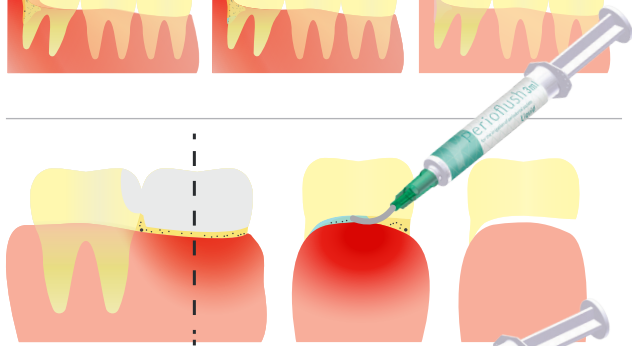

Interdental spaces without proper points of contact

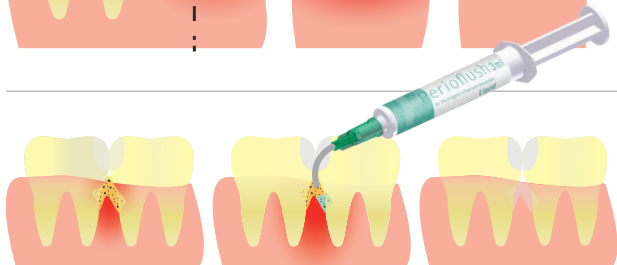

Instruction for use issued on 30/05/2017